**SELECTION BROMSA
WRIST SUPPORT
35208**

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Carefully read this instruction sheet before use!

Indications / Intended Use

This product is intended to restrict movement in the wrist in order to off-load the hand extensors, support the wrist and reduce pain in conditions as lateral epicondylitis, hypermobility, rheumatic problems and unspecific pain. For one persons use only.

Contraindications

Severe edema.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Do not use when product is wet or on wet skin. Do not pull the straps too tight as this can cause poor circulation and/or skin irritation. Check the skin on a daily basis and if you see or feel

any irritations or other problems, remove the product immediately. If you have marks or chafing that do not reduce within a short time (30 minutes) contact your clinician immediately.

Material Specifications

This orthosis is made from Polyester, Lycra, nylon, polyurethane. Stays in PE. This product is not flame retardant. Latexfree.

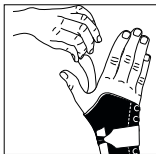
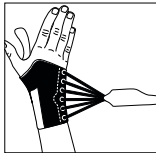
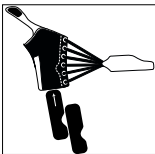
Washing Instruction

Close all Velcro prior to laundering. Remove the plastic stays prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

INSTRUCTION FOR USE

1. Check that the support is complete and without damage.
2. This support has two plastic stays in the pocket. Use one stay for mild resistance and two for increased resistance. Contact the prescriber or fitter when in doubt.
3. Pull the support on like a glove. The stay shall be placed on the back of the hand.
4. Pull the laced strap around the wrist and attach.
5. Secure the thumb strap.
6. Adjust for comfort.

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