

allard^{USA}



INSTRUCTIONS

www.allardusa.com

USER

COMBO
a ToeOFF® Family product

Patent no.:


ToeOFF, ToeOFF Short, ToeOFF Fantasy, ToeOFF NFR, BlueRocker, BlueRocker NFR, KiddieGAIT, KiddieGAIT NFRAU736950, BE1005297, BE114626, DK1005297, DK114626, FI1005297, FI114626, FR1005297, FR114626, IE1005297, IE114626, IT1005297, IT114626, CA2279225, CNZL97181689.1, NL1005297, NLI114626, NO313656, PL194247, CH1005297, CH114626, ES1005297, ES114626, GB1005297, GB114626, SE1005297, SE114626, DE69709184.8-08 DE69732541.5-08, US5897515, ATE210417, ATE289187
Ypsilon, Ypsilon NFR GE60208889.5-08, GB1379201, IT1379201, US6887213

READ THIS INSTRUCTION CAREFULLY BEFORE USE


The brace that you have received is actually two braces in one. The lower part, the ToeOFF® or BlueRocker™ (AFO), and the upper part, the Combo (Thigh cuff with extension control uprights). They should preferably be used together to provide increased ankle stability and/or dynamic walking assist, and help prevent hyperextension of your knee. The lower part (AFO) can also be used as a separate item when desired and authorized by your orthotist and physician.

The Combo part has been shaped and adjusted to fit your leg. You should not make any adjustment to the Combo on your own. If you feel any discomfort or experience any skin irritation, you should immediately remove the brace and contact your orthotist.


PUTTING ON YOUR BRACE

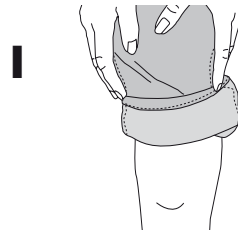

1. Pull trousers or slacks up above your knee.

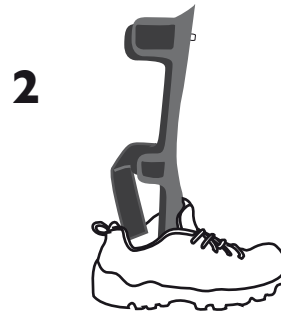


2. Slide the ToeOFF/BlueRocker (the lower part) into your shoe.

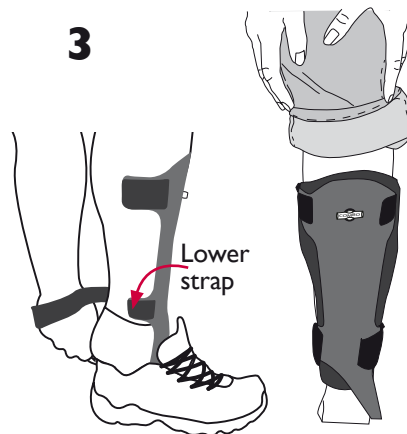


3. Slide your foot into the shoe, attach and wrap the lower Velcro strap only around your calf and attach to the Velcro on the opposite side.

4

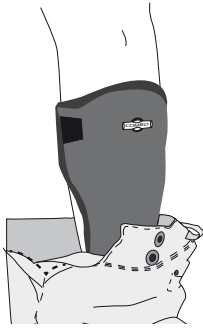

4. Slide your trousers/slacks down below your knee.


5

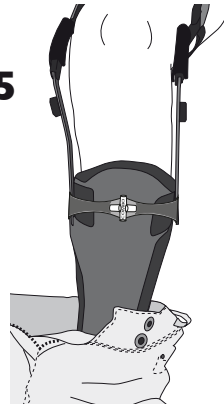

5. Position the thigh cuff on your thigh and connect following the illustrations A-E below.


CONNECTING THE TWO PARTS

6 A

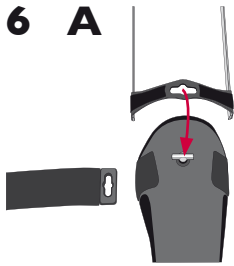


Once the COMBO is fully assembled, then proceed to step B.

B

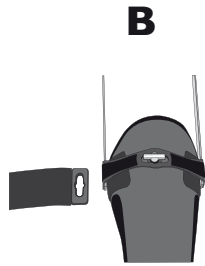


Place the Tibial Connector over the Thumb Screw.

C

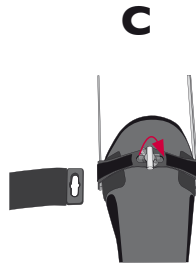


Turn Thumb Screw 90° to the vertical position.

D

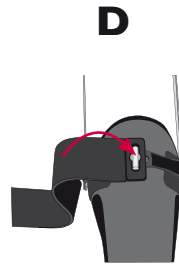


Place rubber molded end of strap over the Thumb Screw.

E

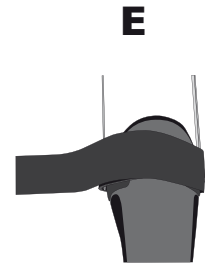

Secure strap completely around the leg so that it overlaps the Thumb Screw.


7

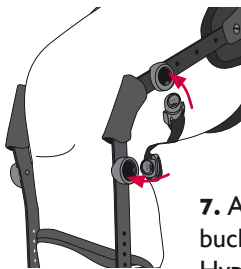

7. Attach the buckles for the Hyperextension Support Pad (soft pad that goes behind your knee).

REMOVING YOUR BRACE

1

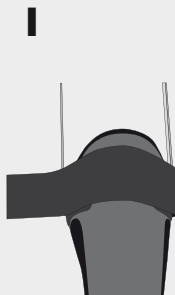

1. Remove the strap that is wrapped around your calf.

2



2. Turn Thumb Screw 90° and remove the strap from the upper section and buckles on one side of the Knee Support Pad.

CLEANING

Both the Combo and the ToeOFF/BlueRocker are padded with a soft interface. This should be removed and washed frequently to avoid skin irritation. Machine wash in lukewarm water (40°C, 104°F) with mild soap, dry flat. Do NOT dry clean, bleach, tumble dry, or iron.

Do not open the C-Shaped Velcro tabs on the Hyperextension Support Padding. Wash the padding with the buckles still in place. Use a washing bag.

SKIN MONITORING

Check the condition of your leg and foot daily. Consult your physician or orthotist immediately in the case of skin irritation or discoloration.

Also read the User Instructions for the ToeOFF/BlueRocker.

I have reviewed and understand these instructions.

Patient Name (Print)

Parent, Caregiver, or Guardian Name

Signature (Patient, Parent, Caregiver, or Guardian) Date _____

Your Combo™ was fitted by:

ATTENTION ORTHOTIST OR PROSTHETIST: Make a copy of these instructions for your records and give original to patient.

Manufactured by:
Camp Scandinavia AB
Karbingatan 38
SE-254 67 Helsingborg
Sweden
Phone +46 42 25 27 01



allard | USA

www.allardusa.com

ALLARD USA, INC.
21 Pine Street, Suite 120
Rockaway, NJ 07866-3130

info@allardusa.com
Toll Free 888-678-6548
Fax 800-289-0809