

## Build a state-of-the art custom KAFO within your own facility today!

*Dynamic Low Profile, Lightweight, Functional Orthotic Solution...for the management of genu-recurvatum or chronic knee instability, accompanied with footdrop.*

Over the years ToeOFF® has proven to be an effective gait rehabilitation orthosis for management of footdrop and ankle instability. It has helped hundreds of thousands of people around the world walk with a closer to normal gait pattern, often being able to return to activities that significantly enhance quality of life. " Genu-recurvatum sometimes accompanies footdrop. Combo is a lightweight, removable knee brace attachment for ToeOFF (or BlueRocker) that creates an innovative orthotic solution for management of footdrop combined with genu-recurvatum.



Low profile and lightweight – a KAFO your patients will wear!

Uprights easily shaped and adjusted to optimum height for patient

Modular components allow you to accommodate variances in thigh and calf circumference

Easy to don and doff

Easily removed when wearer is relaxing (watching TV, etc.)

**COMBO**  
a ToeOFF® Family product



Combo is fitted individually for each patient. The different components can be combined to accommodate leg circumference and length.

**5 COMBO Components (see below) + a ToeOFF or BlueRocker are required to build the custom KAFO:**

1. JOINTS: One size
2. TIBIA CONNECTOR: Generally, select the same size as the ToeOFF/BlueRocker. However, the next smaller size will “snug” the wings of the ToeOFF/BlueRocker closer to the leg for the extra small calf, and the next larger size may better accommodate a larger calf.
3. KNEE CONTROL Component with Popliteal Interface: Take a M-L measurement at the knee (knee width).
4. THIGH CUFF: Take a circumferential measurement at height where thigh cuff is to be located, however, make certain it is not less than 4” (10cm) above the tibial plateau. Take a M-L measurement at same height.
5. THIGH CUFF SOFTKIT: Same size as thigh cuff.

**WARNING:**  
 Weight Limit - 330 pounds (150 kg)  
 Height Limit - 6 1/2 feet (2 m)  
 Use of COMBO with any product other than ToeOFF or BlueRocker is prohibited.

**5 COMPONENTS (+ToeOFF or BlueRocker) REQUIRED TO BUILD A COMBO**

**1 JOINTS WITH UPRIGHTS**

Item No	Size
28780 0000	One Size

**2 TIBIA CONNECTOR**

Item No	Size
28784 0011	S
28784 0012	M
28784 0013	L/XL

**3 KNEE CONTROL COMPONENT WITH POPLITEAL INTERFACE**

Item No	Size	Medial-Lateral	
		cm	inches
28782 0011	S	70 - 105 mm	2 3/4 - 4 1/4 in
28782 0012	M	95 - 130 mm	3 3/4 - 5 1/4 in
28782 0013	L	125 - 160 mm	5 - 6 1/4 in

**4 THIGH CUFF**

Item No. Left	Item No. Right	Size	Circumference		M-L	
			Cm	Inches	Cm	Inches
28781 1011	28781 2011	S	30 - 40 cm	11 3/4 - 15 3/4 in	11 - 15 cm	4 1/4 - 6 in
28781 1012	28781 2012	M	38 - 50 cm	15 - 19 3/4 in	13 - 17 cm	5 1/4 - 6 in
28781 1013	28781 2013	L	48 - 60 cm	19 - 23 3/4 in	15 - 19 cm	6 - 7 1/2 in
28781 1014	28781 2014	XL	58 - 70 cm	22 3/4 - 27 1/2 in	17 - 21 cm	6 3/4 - 8 1/4 in

**5 THIGH CUFF SoftKIT**

Item No	Size (set of 2)
28783 0011	S
28783 0012	M
28783 0013	L
28783 0014	XL