

NEW DESIGN

*softKIT*TM

A padding system for ToeOFF[®] and BlueRockerTM



SoftKITTM in new design

SoftKIT is a padding system for the ToeOFF[®] and BlueRockerTM. In this updated version the number of Micro-Fix parts has been reduced to only 6 and the material is even more soft and durable.

For patients with a protruding tibia we recommend to use the two neoprene pads to create a comfortable channel to avoid pressure on the tibia crest.

Two interfaces are included in each package. Instruct the patient to wash the interface, at least ones a week. Can be machine washed.

softKIT™

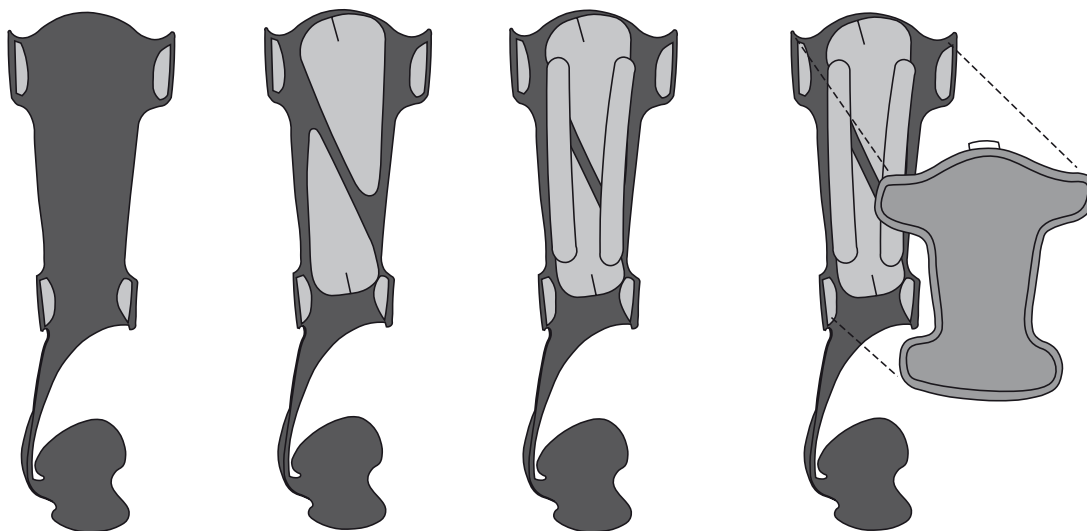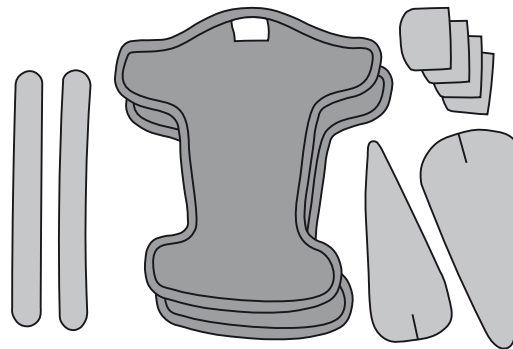

SOFTKIT COMPONENTS

- Two SoftKIT interfaces
- Two vertical neoprene channel pads
- Six MicroFIX patches

SOFTKIT APPLICATION

Start by adhering the larger half-moon shaped MicroFIX (smaller patches on lower wings and larger on the upper wings). Next, place the two teardrop shaped MicroFix in the middle, and then apply the neoprene pads in appropriate location to form channel for the tibia.

Item No.	Description	Sizes
28390	SoftKIT, Complete	XSmall, Small, Medium, Large/XLarge
28396	MicroFIX parts	XS/Small, Medium/Large/XLarge
25040	MicroFIX White	Roll, 50mm x 10m, (2in x 33ft)
25041	MicroFIX Black	Roll, 50mm x 25m (2in x 27yds)