

VISSION™ Fracture Walker

INDICATIONS:

Stable fracture of Talus, Tarsus, Tibia and Fibula, severe to moderate ankle sprain & strains, post-operative immobilization, metatarsal fractures.


CONTRAINDICATIONS:

Proximal Fibular or Tibial and any unstable fractures.


INSTRUCTIONS FOR APPLICATION



Detach all straps and remove liner from the fracture walker.

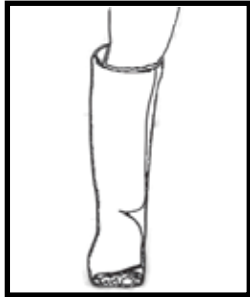


Step into the liner and close snugly by engaging the hook fastener onto the liner surface.

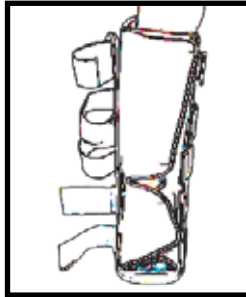

With the liner on, step into the walker.



Ensure the rigid up-rights on the walker are centered over the ankle. Place your heel as far back against the walker as possible to ensure proper fit.

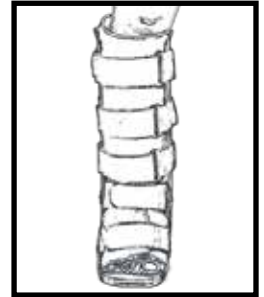

Start with the strap closest to the toes and continue up the leg. Fasten the hook and loop straps assuring a snug fit.

INSTRUCTIONS FOR:

PNEUMATIC APPLICATION



After applying the walker, inflate liner air cell by pressing the bulb on the top front of the boot until the walker feels comfortably secure. Do not over-inflate to the point of restricting blood circulation in your foot.



To deflate the air cell, open the black small cover below the air bulb and press the brass button until all the air escapes from the cell.

8700 Rev C