

allard|USA

ToeOFF[®] SHORT

BLUEROCKER[®] SHORT

New Model!



NEW features

UPDATED STRAP SOLUTION
& PRE-MOUNTED MIKROFIX™!

ToeOFF[®] SHORT BLUEROCKER[®] SHORT

SHORT models are specifically developed to accommodate individuals with a shorter stature, slightly broader shaped tibia, and wider foot at the metatarsal area. Tibia padding should always be used for patient comfort.

Both SHORT models have a heel height of 7 mm in all sizes.

Intended Use

Foot drop in combination with weak plantar flexors, and in some cases limited knee control.

For ToeOFF[®] SHORT: Mild to moderate spasticity and proximal deficits.

For BlueROCKER[®] SHORT: Moderate to severe spasticity and proximal deficits, impairment in multiple leg muscle groups and partial foot amputation. Also for bilateral wearers and balance issues.

Contraindications

Should not be used when patients present with foot and/or leg ulcers, moderate to severe edema, moderate to severe foot deformities, severe proximal deficits or severe spasticity.

More about fitting and product selection can be found in the Allard AFO Professional Instructions.

Item No. ToeOFF [®]	Item No. BlueROCKER [®]	Size	Side	Total height	Footplate length
28990 1011	28995 1011	Small	Left	12"	8 ½"
28990 1012	28995 1012	Medium	Left	12 ¾"	9"
28990 1013	28995 1013	Large	Left	13 ¼"	9 ¾"
28990 2011	28995 2011	Small	Right	12"	8 ½"
28990 2012	28995 2012	Medium	Right	12 ¾"	9"
28990 2013	28995 2013	Large	Right	13 ¼"	9 ¾"
28391 0011	SoftKIT [™] SHORT, Small				
28391 0012	SoftKIT [™] SHORT, Medium				
28391 0013	SoftKIT [™] SHORT, Large				
28768 0011	SHORT Strap set, Small				
28768 0012	SHORT Strap set, Medium				
28768 0013	SHORT Strap set, Large				

