




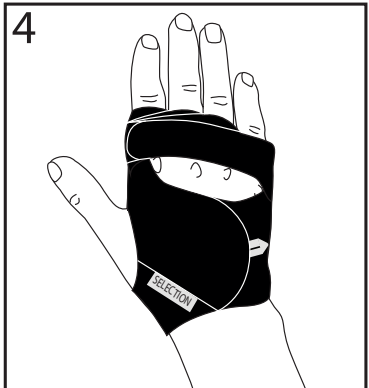
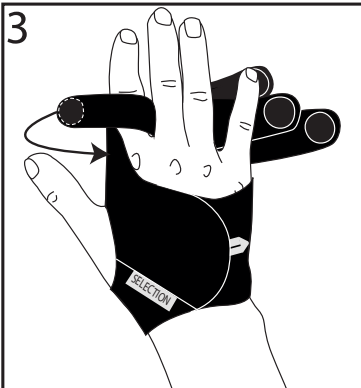
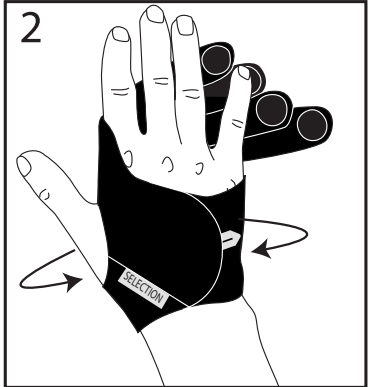
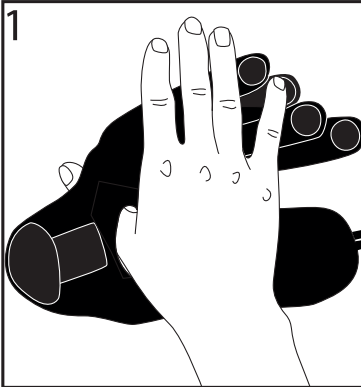
A black and white photograph of a person's hand wearing a black, fabric wrist support device. The device has a strap across the back of the hand and a circular patch on the wrist. The text '35218 SELECTION ULNAR DEVIATION' is printed in white on the strap. A small white tag with the word 'SELECTION' is attached to the wristband. The background is white with a grey wavy border at the top and bottom.

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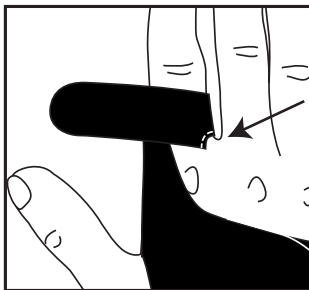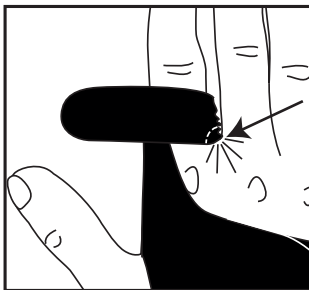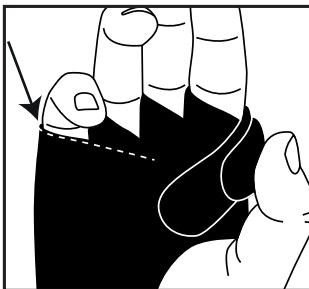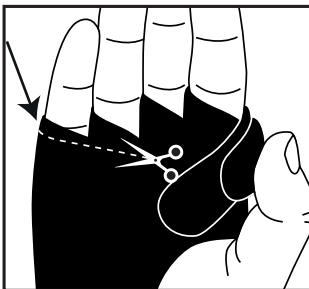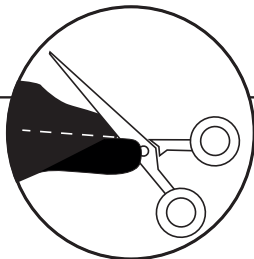ENGLISH

Indications / Intended Use

Textile hand/finger-orthosis for management of Ulnar Deviation. Corrects alignment of joints of the fingers to improve extension, strength and function of the hand and thereby can provide reduced pain and joint stiffness. For patients with Arthritis. Ulnar deviation is likely to be a characteristic for those with various forms of inflammatory diseases, such as Rheumatoid arthritis. Can also be used for realignment and stabilization in people with joint laxity and muscle weakness. Can be combined with a Selection Wrist orthosis.
Single Patient-multiple use.

Contraindications

Fixed contractures.

Adjustments

This product should only be altered by a medical professional person. Minor adjustments can be done during fitting. It's possible to cut in material without frying.

Warnings

Ensure that the straps are not too tight. Do not use when the product is wet or on wet skin.

35218

SELECTION ULNAR DEVIATION

Carefully read this instruction sheet before use!

Check your skin daily for irritation. At first sight of skin irritation remove the orthosis and contact your treating clinician.

Material Specifications

This orthosis is made from nylon, polyester, lycra. This product is not flame retardant. Latex free.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal

Disposal in accordance with local legislation

Information:

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use (See illustrations on page 2-3)

Check that the product is intact and free from damage. Check that you have the correct size and side (L/R).

1. Insert the thumb through the opening.
2. Close the Velcro Strap over the back of the hand.
3. Fasten the strap around the index finger first, then continue with the rest of the fingers.
4. Adjust the straps until the fingers are correctly positioned.
5. If necessary, the orthosis can be customised by trimming with scissors.

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Camp Scandinavia AB, Karbingatan 38