

## S.W.A.S.H.<sup>®</sup> STEADY & GO SIZING GUIDE

<b>PELVIC SECTION</b> Measure: SWASH <sup>®</sup> STEADY - At level of the natural waist for maximum trunk stability. SWASH <sup>®</sup> GO – At the level of the ASIS. Ideal is to fit the orthosis at mid-pelvic girdle to obtain good pelvic stability.	Size	Sizes Available		Circumference	
		STEADY	GO	Adjustment Range	
	1	✓	✓	400-465mm	15 ¼-18 ¼in
2	✓	✓	460-550mm	18-21 ¾in	
3	✓	✓	540-630mm	21 ¼-24 ¾in	
4	✓	✓	620-720mm	24 ½-28 ¼in	
5	N/A	✓	710-810mm	28-32in	
6	N/A	✓	800-900mm	31 ½-35 ½in	

<b>THIGH CUFFS</b> Measure at distal thigh, just proximal to the condyles. Ideal is to fit the cuffs as distal as possible without creating pressure or interference in the popliteal area. However, pressure on the thorax, excessive spinal flexion, or discomfort due to hamstring tightness, may be relieved by moving the cuffs up 1 - 3 inches.	Size	Sizes Available		Circumference	
		STEADY	GO	Adjustment Range	
	1	✓	✓	210-250mm	8 ¼-9 ¾in
2	✓	✓	250-290mm	9 ¾-11 ½in	
3	✓	✓	290-330mm	11 ½-13in	
4	✓	✓	330-380mm	13-15in	
5	✓	✓	380-440mm	15-17 ¼in	
6	✓	✓	440-500mm	17 ¼-19 ½in	


<b>UPRIGHTS</b> Measure waist to mid-patella. At final fitting, should be shortened to level of distal thigh cuff padding. 115° = narrow sitting base 123° = wide sitting base	Size	Upright Diameter		Waist to Mid-Patella	
		STEADY	GO		
	1	6mm	8mm	310mm	12 ¼in
2	8mm	8mm	375mm	14 ¾in	
3	8mm	8mm	440mm	17 ¼in	
4	8mm	8mm	480mm	19in	
5	8mm	8mm	540mm	21 ¼in	
6	8mm	8mm	600mm	23 ½in	

Size		PELVIC STEADY	PELVIC GO	CUFFS	UPRIGHTS 115 Deg.	UPRIGHTS 123 Deg.
1	STEADY	288520001		288540601	288550601	288560601
1	GO		288530001	288540801	288550801	288560801
2	STEADY & GO	288520002	288530002	288540802	288550802	288560802
3	STEADY & GO	288520003	288530003	288540803	288550803	288560803
4	STEADY & GO	288520004	288530004	288540804	288550804	288560804
5	STEADY & GO	N/A	288530005	288540805	288550805	288560805
6	STEADY & GO	N/A	288530006	288540806	288550806	288560806

NOTE: Color Coding indicates interchangeable components.

# S.W.A.S.H.<sup>®</sup>

## STEADY & GO Components

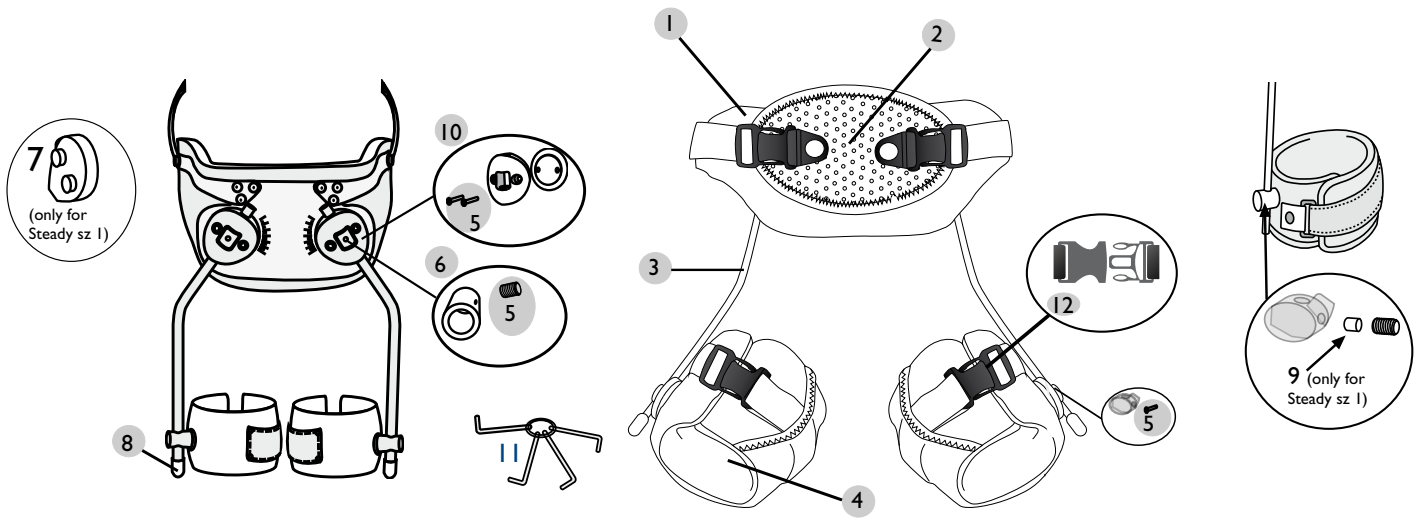

No.	Description	Unit	STEADY	GO
1	Pelvic Padding with Abdominal Pad	1 set	28857	28858
2	Abdominal Pad	1 pcs	28860	28860
3	Upright 6 mm 115°, size 1 only	Pair	28855-06	
3	Upright 8 mm 115°	Pair	28855-08	28855-08
3	Upright 6 mm 123°, size 1 only	Pair	28856-06	
3	Upright 8 mm 123°	Pair	28856-08	28856-08
4	Thigh Cuffs Padding	Pair	28859	28859
5	Screw Kit, size 1		28861-06	
5	Screw Kit, size 2 - 6		28861-08	28861-08
6	Clamp Ring Set, size 1	2 pcs	28862-06	
6	Clamp Ring Set, size 2 - 6	2 pcs	28862-08	28862-08
7	Boss Cover, size 1	2 pcs	28863-06	
8	Upright End Cover, size 1	2 pcs	28864-06	
8	Upright End Cover, size 2 - 6	10 pcs	28864-08	28864-08
9	Brass Bushing, size 1	10 pcs	28865	
10	Hip Joint Set, size 1	2 pcs	28866-06	
10	Hip Joint Set, size 2 - 6	2 pcs	28866-08	28867-08
11	Allen Wrench Key Set	1 set	28868	28868
12	Buckles 25 mm, for size 1 - 3	2 pcs	28869-25	28869-25
12	Buckles 38 mm, for size 4 - 6	2 pcs	28869-38	28869-38

This product should only be fitted by a  
SWASH<sup>®</sup> certified professional.