allard



HIPS at RISK for subluxation?





SITTING Assistant Improve sitting posture - Open up the thorax





GMFCS V

WALKING Assistant

Proprioceptive response helps enhance position sense in space





GMFCS II

Only FUNCTIONAL hip Orthosis that does not block ROM!

S E Z

STANDING Assistant Eliminate Scissoring - Reduce stress on the hips










GMFCS IV


GMFCS IV

HIP Alignment Orthosis

Hip Alignment Benefits


The most visible benefits of the SWASH are readily apparent by improvements in sitting, standing, and if ambulatory, walking posture and stability. However, perhaps the most valuable benefit is guidance for proper hip alignment.

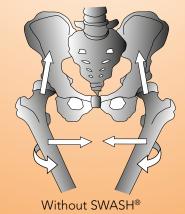

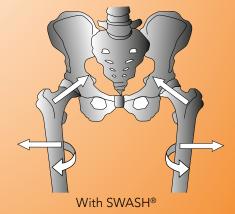

Functional benefits of SWASH

- Dynamic Function allows only biomechanically appropriate motion.
- Transfers adductor tone proximally to achieve more positive postural positioning.
- Proper alignment encourages tri-planar function for proper closed chain muscle function.
- Facilitates the strengthening of muscles to reduce compensations that risk proximal and distal joints.
- Encourages tri-planar function.
- Controls amount of adduction for improved sitting, standing, and if ambulatory, walking functions.
- Encourages proper closed chain muscle function.

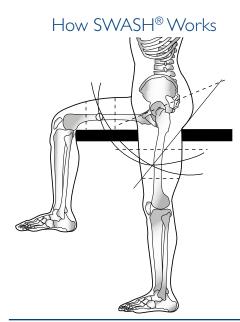

The pathways of motion of the SWASH thigh cuffs mimic the "normal" pathways of motion of the femurs. This is designed into the orthosis through combinations of the angle of the pelvic band in the sagittal plane, and the angle of the hip joint assembles in the transverse plane.

This exerts an external rotary influence on the lower limbs, influencing the knees toward extension and the trunk becomes more erect. As SWASH reaches its limit for adduction, it transfers adductor force proximally to assist posterior rotation of the pelvis to help achieve a more erect posture.



Two models to meet specific needs of your patients! Sizes available to fit infants to small adults





When maximum trunk control is required, i.e., candidates who lack muscle strength or upper body control to sit upright.

When patient is primarily non-ambulatory (GMFCS IV-V).

When the greater pelvic coverage area triggers more desirable neurosensory motor response.





When primary goal is to control scissoring gait.

When patient has limited space between the iliac crest and the rib cage.

Patient is ambulatory with or without assistive devices and does not require the additional torso support offered with SWASH Steady.

IMPROVED MUSCLE FUNCTION = IMPROVED OUTCOMES









Allard USA expresses their deepest gratitude and appreciation to those who helped to make this presentation possible: All the beautiful children, their parents and siblings and allowed us to share their photos; OrthoCare Services, Moore, Oklahoma; The Children's Center Rehabilitation Hospital, Bethany, OK; Photos by Keshia, Keshia Kastl, OKC, OK.

Contact your local certified SWASH O&P Provider for in-service or to participate in your next CP Clinic for trial fittings on potential candidates.

Note: If no provider listed above, call Allard USA Customer Service 888-678-6548 or email info@allardusa.com.

