CROSS™

28715
HYPEREXTENSION
KNEE BRACE

patent pending

Intended use

The brace is intended for mild to moderate knee hyperextension. The brace can also be used together with Allard AFOs when knee hyperextension is present together with foot drop. Intended for one person use.

Indications

Knee hyperextension due to Stroke, Post-Polio or other neuromuscular diseases or injuries like weakening M. Quadriceps or weak calf muscles. Genu recurvatum with intact ML ligaments.

Contraindications

The brace is not intended for sport activity. The brace should not be used when all function in M. Quadriceps is lost or when knee ligaments are disabled. Hyperextension ≥ 15 degrees.

Adjustments

This product should only be altered by a medical professional.

Warnings

Do not use when product is wet or on wet skin. Do not pull the straps too tight as this can cause poor circulation and/or skin irritation. Check the skin on a daily basis and if you see or feel any irritations or other problems, remove the product

immediately. If you have marks or chafing that do not reduce within a short time (30 minutes) contact your clinician immediately. The only approved combination is with an Allard AFO.

Material Specifications

This product is made from Nylon, Lycra, Polyester, Aluminium and Polyurethane. Does not contain Latex. This product is not flame retardant.

Washing Instruction

Remove stays before wash. Close all Velcro straps prior to laundering. Use a washing bag. Machine wash in 40°C (104°F) gentle cycle. Dry flat. Do not tumble dry or dry clean.



Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Use the attached graphic instruction sheet to instruct the user to put on and remove. Make sure the user understands the instructions and can self-apply and remove.
3. The joints can be set in various degrees of range-of-motion. Various stops are included in the package. Make sure that both joints have equal setting.
4. Instruct the user how to remove and replace the joints. Each upright is labeled with up/out. Show to the user to ensure that the uprights will be positioned correctly after laundering.
5. Give both this instruction and the pictogram to the user/caregiver after fitting.

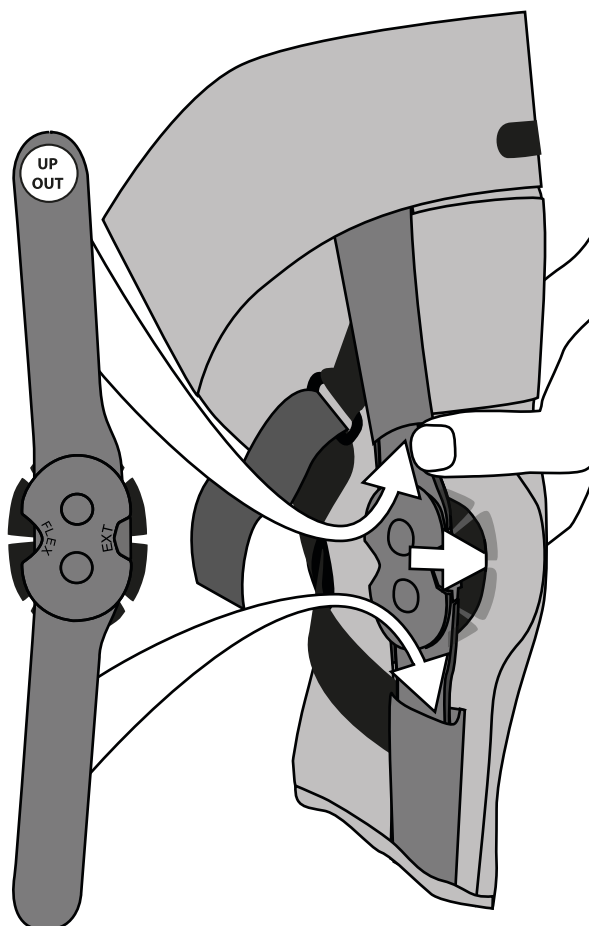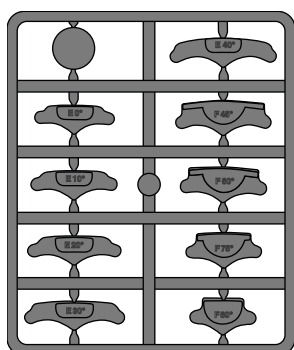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