

CSUS


by Allard USA



HINGED KNEE SLEEVE
Rx Only — Single Patient Use Only



Instructions for Use

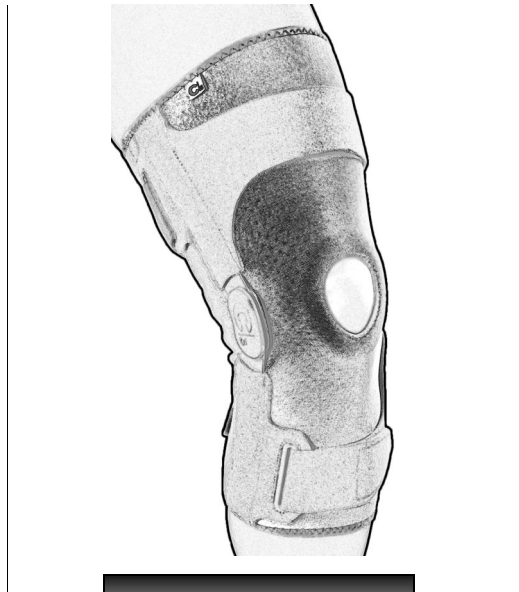

FEEL THE FUNCTION

The intended function of this sleeve is to provide gentle support and medial/lateral stability through compression for the treatment of mild knee instabilities. Review instructions on back page carefully before application.

For questions, please contact your physician, the professional who fit this device or Allard USA Customer Service:

CALL: 888-678-6548 FAX: 800-289-0809
info@allardusa.com

Allard USA, Inc. 300 Forge Way, Suite 3 Rockaway, NJ 07866-2056
www.allardusa.com

CSUS


by Allard USA



HINGED KNEE SLEEVE
Rx Only — Single Patient Use Only



Instructions for Use

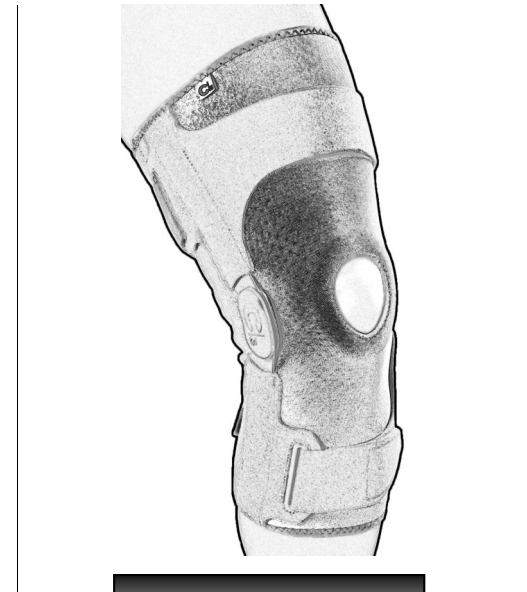

FEEL THE FUNCTION


The intended function of this sleeve is to provide gentle support and medial/lateral stability through compression for the treatment of mild knee instabilities. Review instructions on back page carefully before application.

For questions, please contact your physician, the professional who fit this device or Allard USA Customer Service:


CALL: 888-678-6548 FAX: 800-289-0809
info@allardusa.com

Allard USA, Inc. 300 Forge Way, Suite 3 Rockaway, NJ 07866-2056
www.allardusa.com


Hinged Knee Sleeve

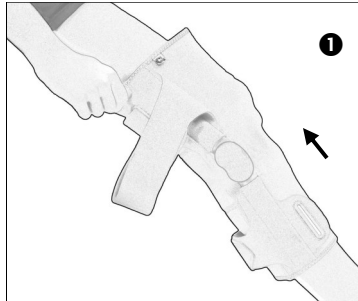


1. With logo tag up, pull sleeve onto leg. Make sure knee cap is centered in opening.

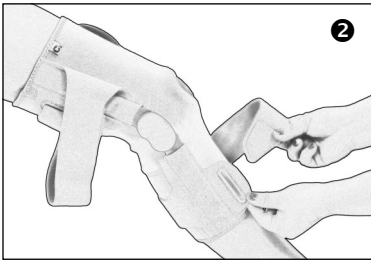

2. Attach posterior calf strap, then anterior calf strap (shown).

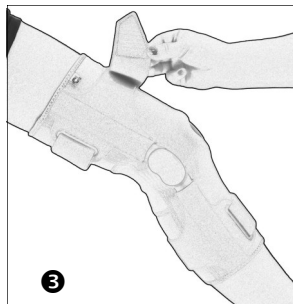

3. Attach posterior thigh strap, then anterior thigh strap (shown). Adjust straps as needed to secure hinges in place along sides of leg.

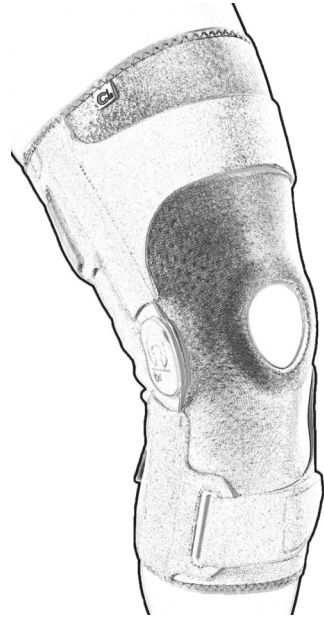

Indications / Intended Use

Sprains and strains; Tendonitis; Arthritis; Bursitis; Cruciate ligament, meniscal and/or collateral ligament injuries.

Contraindications

Severe knee instabilities.

Adjustment

This brace should only be fitted by a licensed medical professional.


Warning

Do not overuse as this can cause skin irritation. If you feel any irritations or other problems, remove sleeve. Check skin on a daily basis.


Wash instruction

Hand wash in cold water using mild detergent. Air dry.


Hinged Knee Sleeve

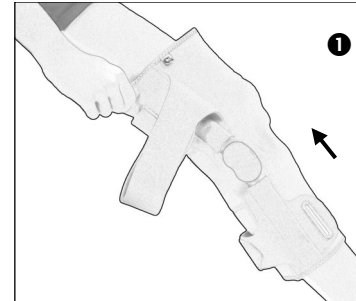


1. With logo tag up, pull sleeve onto leg. Make sure knee cap is centered in opening.

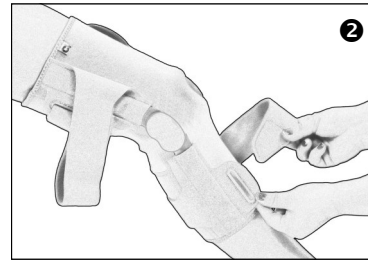

2. Attach posterior calf strap, then anterior calf strap (shown).

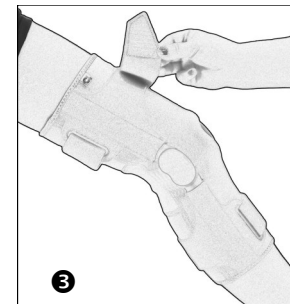

3. Attach posterior thigh strap, then anterior thigh strap (shown). Adjust straps as needed to secure hinges in place along sides of leg.

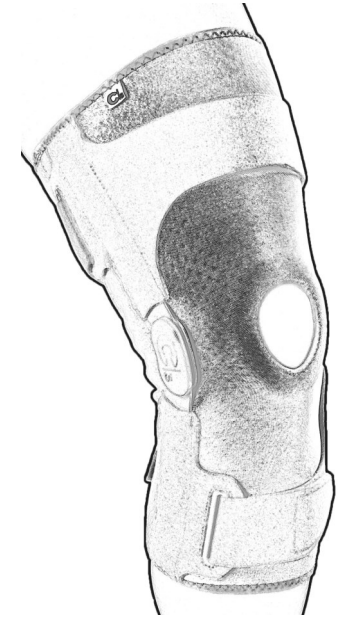

Indications / Intended Use

Sprains and strains; Tendonitis; Arthritis; Bursitis; Cruciate ligament, meniscal and/or collateral ligament injuries.

Contraindications

Severe knee instabilities.

Adjustment

This brace should only be fitted by a licensed medical professional.

Warning

Do not overuse as this can cause skin irritation. If you feel any irritations or other problems, remove sleeve. Check skin on a daily basis.

Wash instruction

Hand wash in cold water using mild detergent. Air dry.