

CSUS

by Allard USA

C|s

UNIVERSAL KNEE IMMOBILIZER

Rx Only — Single Patient Use Only

C|s

Instructions for Use



FEEL THE FUNCTION

The intended function of this brace is to immobilize the knee joint post operatively or after injury. Review instructions on back page carefully before application.

For questions, please contact your physician, the professional who fit this device or Allard USA Customer Service:

CALL: 888-678-6548 FAX: 800-289-0809
info@allardusa.com

Allard USA, Inc. 300 Forge Way, Suite 3 Rockaway, NJ 07866-2056
www.allardusa.com

CSUS

by Allard USA

C|s

UNIVERSAL KNEE IMMOBILIZER

Rx Only — Single Patient Use Only

C|s

Instructions for Use



FEEL THE FUNCTION


The intended function of this brace is to immobilize the knee joint post operatively or after injury. Review instructions on back page carefully before application.

For questions, please contact your physician, the professional who fit this device or Allard USA Customer Service:

CALL: 888-678-6548 FAX: 800-289-0809
info@allardusa.com



Allard USA, Inc. 300 Forge Way, Suite 3 Rockaway, NJ 07866-2056
www.allardusa.com

Universal Knee Immobilizer

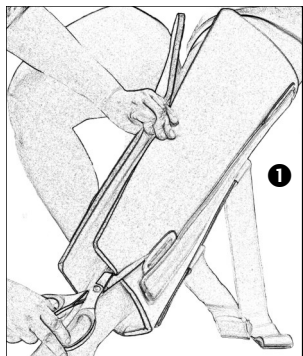

1. Detach the straps and side panels. Wrap foam around leg and hold as shown. Move side panels to medial and lateral positions along leg. Trim excess foam for close fit.

2. Starting at the ankle, feed inelastic straps through buckles and secure. Make sure the brace is centered over knee.

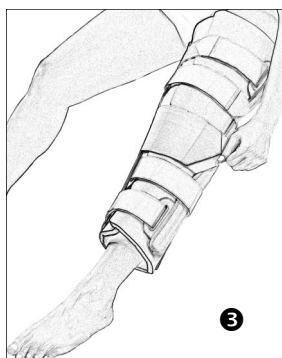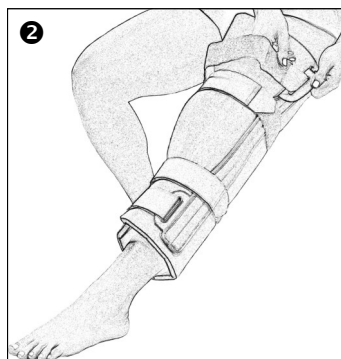

3. Finish by wrapping elastic strap over knee and securing. Adjust straps as needed.

Indications / Intended Use

Post operative Immobilization,
Grade II or III ligament sprains.

Contraindications

Diabetes, skin sensitivity,
poor peripheral circulation.

Adjustment

This immobilizer should only be fitted by a licensed medical professional.


Warning

Do not overtighten, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, remove brace. Check skin on a daily basis.

Wash instruction



Hand wash in cold water using
mild detergent. Air dry.

Universal Knee Immobilizer

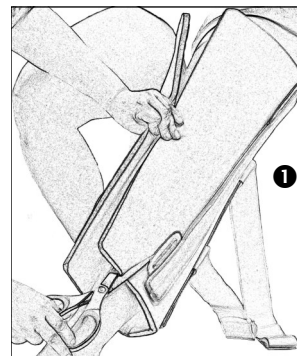

1. Detach the straps and side panels. Wrap foam around leg and hold as shown. Move side panels to medial and lateral positions along leg. Trim excess foam for close fit.

2. Starting at the ankle, feed inelastic straps through buckles and secure. Make sure the brace is centered over knee.

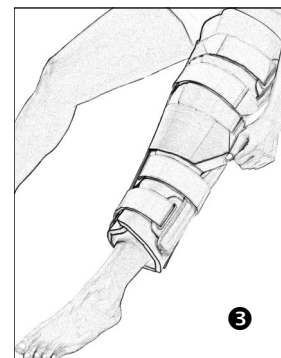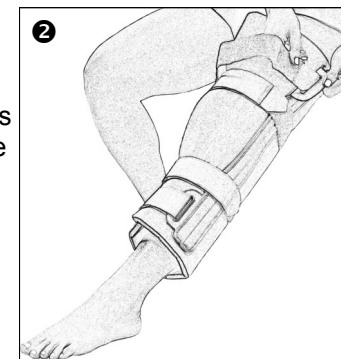

3. Finish by wrapping elastic strap over knee and securing. Adjust straps as needed.

Indications / Intended Use

Post operative Immobilization,
Grade II or III ligament sprains.

Contraindications

Diabetes, skin sensitivity,
poor peripheral circulation.

Adjustment

This immobilizer should only be fitted by a licensed medical professional.

Warning

Do not overtighten, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, remove brace. Check skin on a daily basis.

Wash instruction

Hand wash in cold water using
mild detergent. Air dry.