

CSUS

by Allard USA

C|s

KNEE SLEEVE
Rx Only — Single Patient Use Only

C|s

Instructions for Use



FEEL THE FUNCTION

The intended function of this sleeve is to provide gentle support through compression for the treatment of mild knee instabilities. Review instructions on back page carefully before application.

For questions, please contact your physician, the professional who fit this device or Allard USA Customer Service:

CALL: 888-678-6548 FAX: 800-289-0809
info@allardusa.com

Allard USA, Inc. 300 Forge Way, Suite 3 Rockaway, NJ 07866-2056
www.allardusa.com

CSUS

by Allard USA

C|s

KNEE SLEEVE
Rx Only — Single Patient Use Only

C|s

Instructions for Use



FEEL THE FUNCTION

The intended function of this sleeve is to provide gentle support through compression for the treatment of mild knee instabilities. Review instructions on back page carefully before application.


For questions, please contact your physician, the professional who fit this device or Allard USA Customer Service:

CALL: 888-678-6548 FAX: 800-289-0809
info@allardusa.com

Allard USA, Inc. 300 Forge Way, Suite 3 Rockaway, NJ 07866-2056
www.allardusa.com

Knee Sleeve

With logo tag up, pull sleeve onto leg. Make sure knee cap is centered in opening.

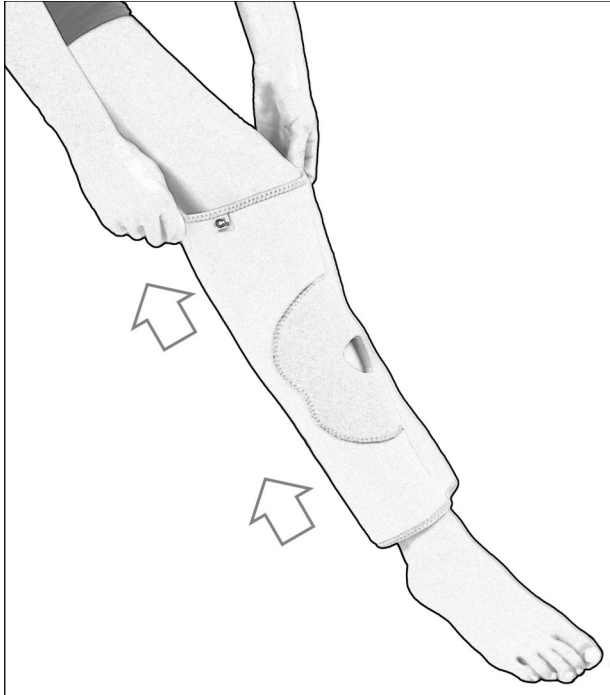

Indications / Intended Use

Post-operative rehabilitation; Sprains and strains; Tendonitis; Arthritis; Bursitis

Contraindications

Severe knee instabilities.

Adjustment

This sleeve should only be fitted by a licensed medical professional.

Warning


Do not overuse as this can cause skin irritation. If you feel any irritations or other problems, remove sleeve. Check skin on a daily basis.

Wash instruction

Hand wash in cold water using mild detergent. Air dry.

Knee Sleeve

With logo tag up, pull sleeve onto leg. Make sure knee cap is centered in opening.

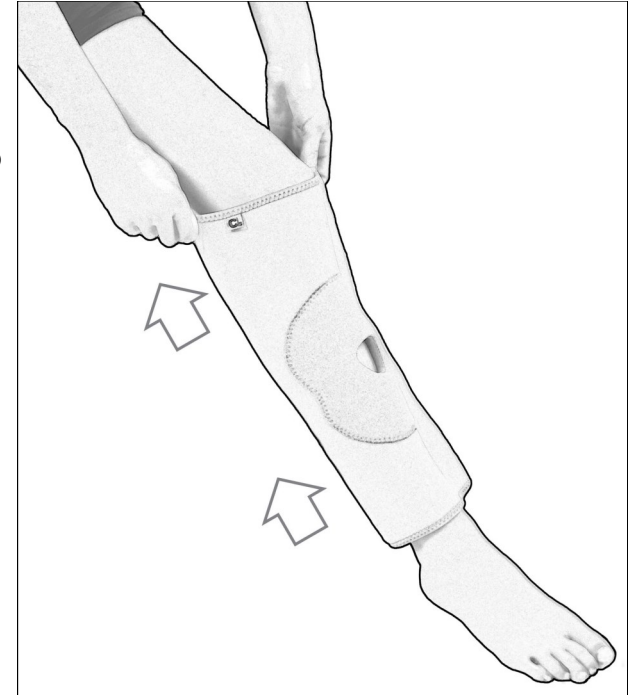

Indications / Intended Use

Post-operative rehabilitation; Sprains and strains; Tendonitis; Arthritis; Bursitis

Contraindications

Severe knee instabilities.

Adjustment

This sleeve should only be fitted by a licensed medical professional.

Warning

Do not overuse as this can cause skin irritation. If you feel any irritations or other problems, remove sleeve. Check skin on a daily basis.

Wash instruction

Hand wash in cold water using mild detergent. Air dry.