


KiddieGAIT®

Start off on the right foot - give toddlers an orthosis that will allow their little feet to move in a more fluid and natural biomechanical gait pattern. Carbon composite with anterior shell, lateral strut, and an engineered footplate with open calcaneus. Designed to allow integration of your custom foot orthotic to help control ground-up forces. Anterior shell to assist in management of proximal deficits by helping to manage either knee hyperextension or crouch gait. KiddieGAIT® should always be worn with a soft interface between the leg and the anterior shell. A non-skid interface attached to the footplate will help prevent any plastic orthotic addition from damaging the carbon composite footplate (sold separately). SoftKIT™ interfaces and SoftSHELLs are also sold separately. KiddieGAIT® comes with D-ring straps. Straps without D-ring closures are available as a separate purchase. See "Accessories/Replacement Parts" for more information on these additional options. More about product selection and customization can be found in the Allard AFO Professional Instructions.

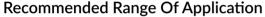

Foot drop, gait deviation secondary to proprioceptive deficit (either unstable or low-tone gait), toe-walker with no mid-foot collapse, low-tone crouch gait, Spina Bifida, Cerebral Palsy, Muscular Dystrophy, MMC.

Contraindications

Lacking ROM towards dorsiflexion (need at least 5°dorsiflexion past neutral), very rigid foot structure, quadriceps spasticity, fixed postural genu valgum or genu varum, fixed postural pes valgus or pes varus, knee hyperextension that can not be orthotically managed.

,	<	7
(ľ)
	٥	1
•	C	5
	C	5
	`	2
-	_	



Item No.	Product Description	Size	Footplate Length	Height
283721008	KiddieGAIT®	BabySmall	4 1/4in	6in
283721009	KiddieGAIT®	BabyMedium	5in	7in
283721010	KiddieGAIT®	BabyLarge	5 1/2in	7 3/4in
289011011	KiddieGAIT®	Small	6 1/4in	8 5/8in
289011012	KiddieGAIT®	Medium	7 1/16in	10 1/8in
289011013	KiddieGAIT®	Large	7 7/8in	11 5/8in
289011014	KiddieGAIT®	X-Large	8 1/4in	12 3/8in
283722008	KiddieGAIT®	BabySmall	4 1/4in	6in
283722009	KiddieGAIT®	BabyMedium	5in	7in
283722010	KiddieGAIT®	BabyLarge	5 1/2in	7 3/4in
289012011	KiddieGAIT®	Small	6 1/4in	8 5/8in
289012012	KiddieGAIT®	Medium	7 1/16in	10 1/8in
289012013	KiddieGAIT®	Large	7 7/8in	11 5/8in
289012014	KiddieGAIT®	X-Large	8 1/4in	12 3/8in