

KiddieFLOW™

KiddieFLOW™ bridges the gap between insoles/SMOs and our current KiddieGAIT® and KiddieROCKER®. KiddieFLOW™ allows for better control of foot positioning in late swing which, in turn, aids stability during stance. It also gives dorsiflexion assistance in the swing phase, while allowing for more range of motion in stance phase.

KiddieFLOW™ should always be worn with a soft interface between the leg and the anterior shell. A non-skid interface attached to the footplate will help prevent any plastic orthotic addition from damaging the carbon composite footplate. SoftKIT™ interfaces and SoftSHELLs are sold separately. (See "Accessories/Replacement Parts" for more information.) More about product selection and customization can be found in the Allard AFO Professional Instructions.

Recommended Range Of Application

Foot drop, gait deviation secondary to proprioceptive deficit (either unstable or low-tone gait), toe-walker with no mid-foot collapse, low-tone crouch gait, Spina Bifida, Cerebral Palsy, Muscular Dystrophy, MMC.

Contraindications

Lacking ROM towards dorsiflexion (need at least 5° dorsiflexion past neutral), very rigid foot structure, quadriceps spasticity, fixed postural genu valgum or genu varum, fixed postural pes valgus or pes varus. Knee hyperextension that can not be orthotically managed.



Item No.	Product Description	Size	L/R	Length	Height
288921011	KiddieFLOW™	Small	Left	6 1/4in	8 5/8in
288921012	KiddieFLOW™	Medium	Left	7 1/16in	10 1/8in
288921013	KiddieFLOW™	Large	Left	7 7/8in	11 5/8in
288921014	KiddieFLOW™	X-Large	Left	8 1/4in	12 3/8in
288922011	KiddieFLOW™	Small	Right	6 1/4in	8 5/8in
288922012	KiddieFLOW™	Medium	Right	7 1/16in	10 1/8in
288922013	KiddieFLOW™	Large	Right	7 7/8in	11 5/8in
288922014	KiddieFLOW™	X-Large	Right	8 1/4in	12 3/8in