

# ToeUP™ Textile Ankle Foot Orthosis

ToeUP is a textile ankle foot orthosis designed for individuals with a mild form of foot drop. This orthosis features a low-profile textile ankle section that fastens above the malleoli (the bony prominence of the ankle). The ankle section is connected using an elastic cord with hooks, which are secured beneath the lacing or within the lacing holes on the shoe. This mechanism provides elevation for the forefoot during ambulation. You can adjust the level of elevation by altering the placement of the hooks or the strap on the ankle section. Additionally, there is a supplementary D-ring to fasten the straps together. ToeUP also offers an optional foot part component for indoor use. It is important to use ToeUP in conjunction with a sock.



## Recommended Range Of Application

For patients experiencing mild symptoms of foot drop due to neuro- and/or muscular weakness in dorsiflexion.

## Contraindications

Should not be use by patients exhibiting open wounds, swelling, redness, or a significant buildup of heat, as well as severe oedema and persons sensitive to Latex.

## Measurements

Measurement 1: Circumference of the ankle

Measurement 2: Circumference around foot

Item No.	Product Description	Size	L/R	Measurement 2	Measurement 1
285012013	ToeUP Foot Part	Large	Right	9.84" – 11.42" (25-29 cm)	-
285060013	ToeUP Ankle Part	Large	Bilateral	-	9.84" – 11.42" (25-29 cm)
285060012	ToeUP Ankle Part	Medium	Bilateral	-	8.27" – 9.84" (21-25 cm)
285060011	ToeUP Ankle Part	Small	Bilateral	-	6.69" – 8.27" (17-21 cm)
285060010	ToeUP Ankle Part	X-Small	Bilateral	-	5.12" – 6.69" (13-17 cm)
285050000	Under Lace Hook	-	-	-	-
285040000	ToeUP D-Ring	-	-	-	-
285030000	ToeUP Hooks	-	-	-	-
285020013	ToeUP Elastic Cord	Medium - Large	-	-	-
285020011	ToeUP Elastic Cord	X-Small - Small	-	-	-
285000010	ToeUP	X-Small	Bilateral	-	5.12" – 6.69" (13-17cm)
285012012	ToeUP Foot Part	Medium	Right	8.27" – 9.84" (21-25 cm)	-
285012011	ToeUP Foot Part	Small	Right	6.69" – 8.27" (17-21 cm)	-
285011013	ToeUP Foot Part	Large	Left	9.84" – 11.42" (25-29 cm)	-
285011012	ToeUP Foot Part	Medium	Left	8.27" – 9.84" (21-25 cm)	-
285011011	ToeUP Foot Part	Small	Left	6.69" – 8.27" (17-21 cm)	-
285000013	ToeUP	Large	Bilateral	-	9.84" – 11.42" (25-29 cm)
285000012	ToeUP	Medium	Bilateral	-	8.27" – 9.84" (21-25 cm)
285000011	ToeUP	Small	Bilateral	-	6.69" – 8.27" (17-21 cm)