


Lateral Wedge & Heel Wedges

WedgeKIT

Pressure Sensitive Adhesive (PSA) Backed EVA Foam

Item # 28766

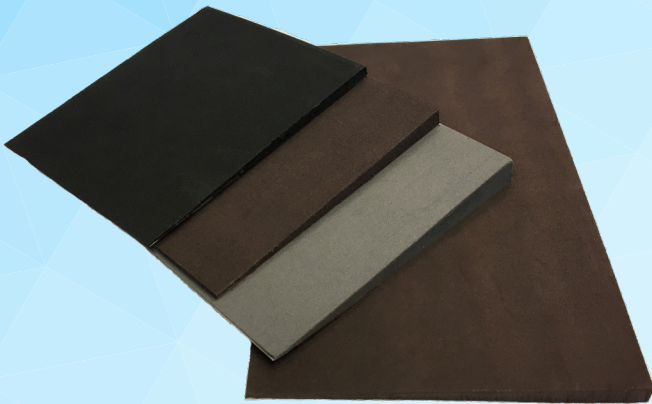

Posting wedges are used for correcting or adjusting medial & lateral slopes.

EVA Color	Dimension	Slope
Gray	3.875"	10 mm - 1 mm
Chocolate	3.875"	7 mm - 1 mm
Black	3.875"	5 mm - 1 mm
Chocolate	7.75"	7 mm - 1 mm

All wedges are Shore A60 Durometer

WedgeKIT

Application Instructions

allard|USA

Preparation

Ensure that application area is clean of dirt, dust, or any other foreign object that may impede adhesion. The recommended application temperature is between 60°F and 120°F. Using a heat gun to slightly warm the application area will help to ensure a better bond, but is not necessary.

Application

Lateral Wedge - Once you have chosen the position of the lateral post/wedge, peel away backing. Begin application by starting from either end and working down the length of the arch support. Keep a slight bend in the posting strip and use a rolling action to keep air bubbles out. After laying the strip down, apply pressure with hands throughout from both sides to secure the bond. Trim excess to match the shape of the device using a knife or shears.

Heel Wedge - Once you have chosen appropriate slope and position of the heel post/wedge, peel away backing. Begin application by starting from the center of the thin edge of the posting wedge. Work toward the deepest part of the heel and then outwards. Try to make full contact with the device to avoid air pockets. Apply pressure with hands throughout from both sides to secure the bond. Trim excess to match the shape of the device using a knife or shears.

Warranty

We guarantee the quality of our materials and products. If a material or product is defective, please contact Allard USA's Customer Service at 888-678-6548, Option 1 to obtain a Return Authorization (RA) #. When received we will inspect the item(s) and if found defective, we will either replace the item or refund the purchase price. This warranty does not apply to adjustments incidental to the wearing of an item, adjustments due to physical changes of the wearer, misuse of a product, or products that have been altered by anyone other than Allard USA.

Assembled in the USA

E300.1M0118

allard|USA

allardusa.com

ALLARD USA, INC.
300 Forge Way, Suite 3
Rockaway, NJ 07866-2056

info@allardusa.com
Toll Free: 888-678-6548
Fax: 800-289-0809